

**Perfect high gloss!
Only one firing!**



K2 Glaze paste GL
for pressable ceramics and
for milled monolithic
zircon works


Step 1

Colorisation of milled object

DIP & BRUSH

K2 Zircon liquid

397-5001/5019

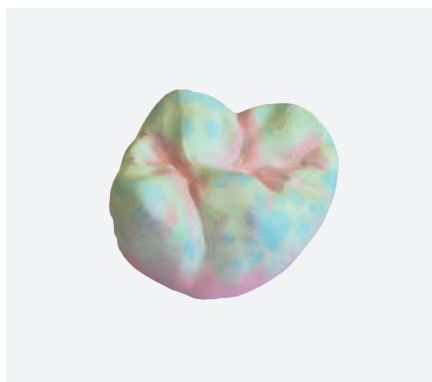

Immerse the milled zirconia restorations in proper dentin colour or individualize them with dentin and enamel colours. Sinterize them.

Step 2

Staining of sinterized object

K2 Stain Zircon Paints

340-0201/0210



Stain Humidor # 312-0001

Step 3


Apply the Glaze paste

K2 Glaze paste GL for pressable ceramics and milled monolithic zircon works

Content: 5g

336-0010

**Perfect high gloss!
Only one firing!**

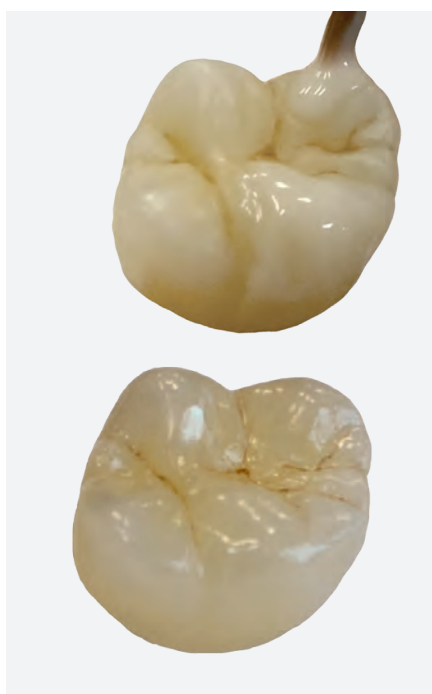

Apply Glaze paste which contains microfine ceramics particles onto colorized and sinterized object. Make a firing

1. Take a **clean container** or Stain Humidor, put there a small quantity of Glaze paste and mix it well together with a Glaze liquid till **gel-like consistency**.
 - more liquid = less gloss
 - less liquid = more gloss

Never bring in contact with water!

2. Apply a **thick layer** of the mixture onto the object
3. **Slightly vibrate** for even spread of the mixture
4. Glaze firing in accordance with zircon manufacture's instructions

During the storage the Glaze paste may dry out. It will have no influence onto the quality.



Caratissima O/B # 312-4000

K2 Zircon Glaze Fluid GF
386-0025/50



Yeti Dentalprodukte GmbH · Industriestrasse 3 · D-78234 Engen
www.yeti-dental.com · e-mail: info@yeti-dental.com